


Wombat Living measurement guide

A simple guide to help you choose the correct size Wombat Living for your child.

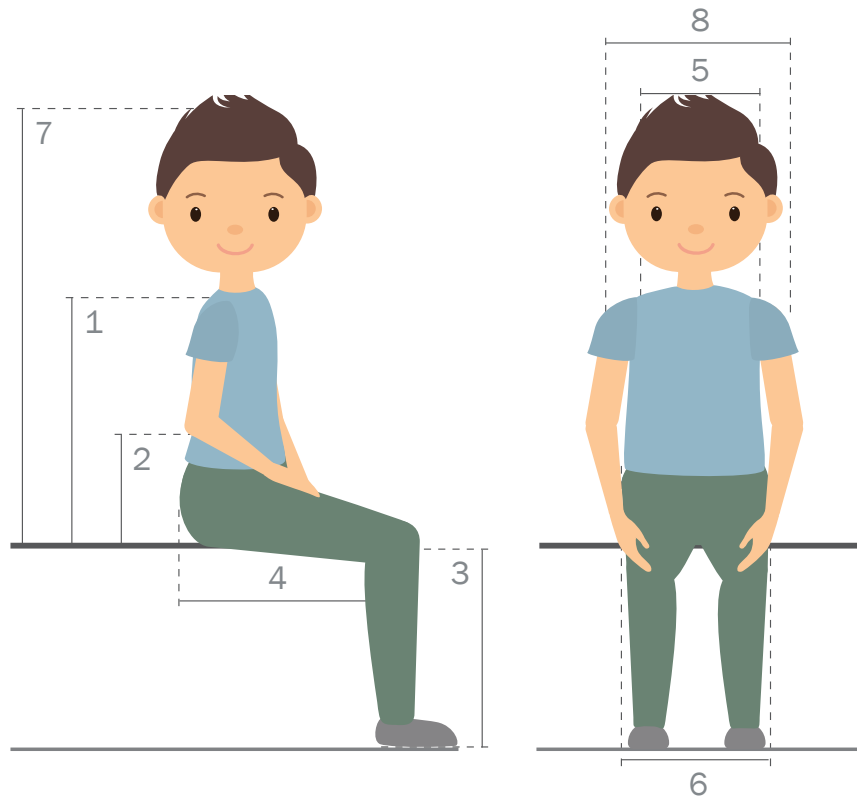

This is a simple guide to make sure the chair is correctly configured for the user. To measure accurately, always make sure that the user is lying on a firm flat surface with hips, knees and ankles at a 90-degree angle.

Collect child's measurements

| No. | Type | Measurement |
|-----|---------------------------------|-------------|
| 1 | Back height | |
| 2 | Arm height | |
| 3 | Lower leg length | |
| 4 | Seat depth | |
| 5 | Chest width | |
| 6 | Seat width | |
| 7 | Height from seat to top of head | |
| 8 | Shoulder width | |

Example on how to use the guide

Step 1: Collect child's measurements

Back height.....350 mm
 Arm height.....130 mm
 Lower leg length.....240 mm
 Seat depth.....270 mm
 Chest width.....200 mm
 Seat width.....220 mm
 Height from seat to top of head.....550 mm
 Shoulder width.....270 mm

Step 2: Choose size

You now have to choose the right size for your child. We recorded Dani's measurements (above) and using the measurement chart below, we could conclude that the optimal size for Dani was an Wombat Living size 2. This size provided support for Dani, and still Dani is able to grow several centimeters in height.

Step 3: Double check measurements

Then you should compare the measurements from the child with the measurements from the Wombat Living in the chart below.



R82 Wombat Living measurements / find the right size for your child

| | Size 1 Gas & Power | Size 2 Gas & Power | Size 3 Gas & power | Size 3 Hydraulic |
|-----------------------------------|-----------------------|-----------------------|-----------------------|---------------------|
| Backrest height | 280 - 340 mm | 310 - 380 mm | 390 - 475 mm | 390 - 475 mm |
| Effective seat width | 300 mm | 370 mm | 420 mm | 420 mm |
| Effective seat depth | 180 - 310 mm | 250 - 380 mm | 320 - 500 mm | 320 - 500 mm |
| Seat surface height at front edge | 230 - 610 mm | 230 - 610 mm | 330 - 630 mm | 390 - 630 mm |
| Max. user mass | 45 kg | 45 kg | 70 kg | 70 kg |

2